

Remote setup submission instructions


To submit your STL file, please follow the instructions below.

Initial setup

Ensure InciseCAD has been installed using the customer serial key supplied.

Submitting your STL



Run InciseCAD and select the  icon from the top toolbar. Input the prescription details and click “OK”.

Create Prescription

Prescription Reference	<input type="text" value="framework name"/>
Dentist Name	<input type="text"/>
Patient	<input type="text" value="Renishaw "/>
Patient Fit Date	<input type="text" value="24/11/2015"/>

Notes:

“Prescription Reference” will be used throughout manufacturing to identify your part. Please ensure it is traceable for your own records and to your patient’s file.


The “Patient” field should contain anonymised personal data. It is your responsibility to obtain your patient’s written consent to disclose their name or any other personal information, if you do not adhere to our anonymisation guideline.

Select your prescription (now displayed on the left-hand side of the screen).




Select the  icon from the top toolbar and set the material.

Select framework configuration from the image on the right-hand side of the screen and then select “Anatomical Units” followed by “Anatomical Coping / Anatomical Pontic”. Click “OK”.

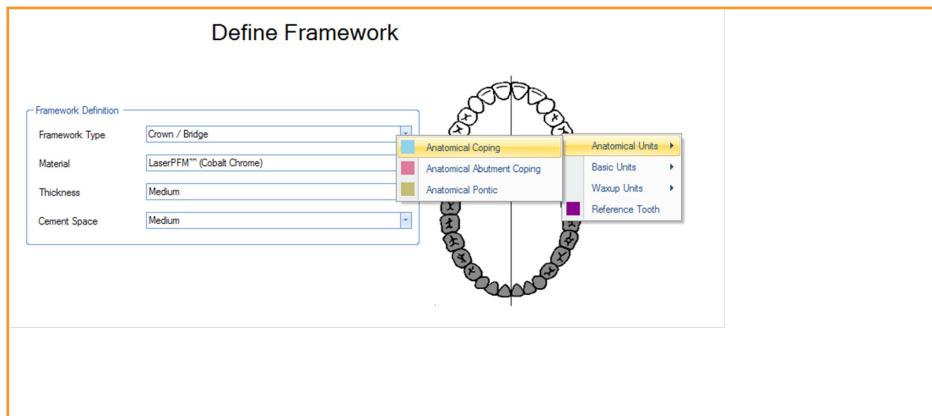

Select the  icon from the top toolbar and upload your STL file.



Select the  icon from the top toolbar and click “OK”.

Your design will now be submitted to Renishaw for manufacturing. If you have problems uploading your STL file at this stage, please contact Renishaw Dental Support on +44 (0) 1453 524543.