


LaserRPD™ setup submission instructions

Thank you for choosing Renishaw for the manufacture of your removable partial denture (RPD) framework. To submit your design for manufacture, please follow the instructions below.


Initial setup

CAUTION: Ensure InciseCAD is not running during the following steps.

Select “Configuration” from the “incise CAD” folder within your Windows start menu.

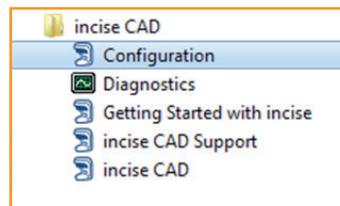


Select the “Additional Serial Keys” tab and input the name as “LaserRPD”. Enter the serial key you have been provided with, click the “Add” button and then select “OK”.

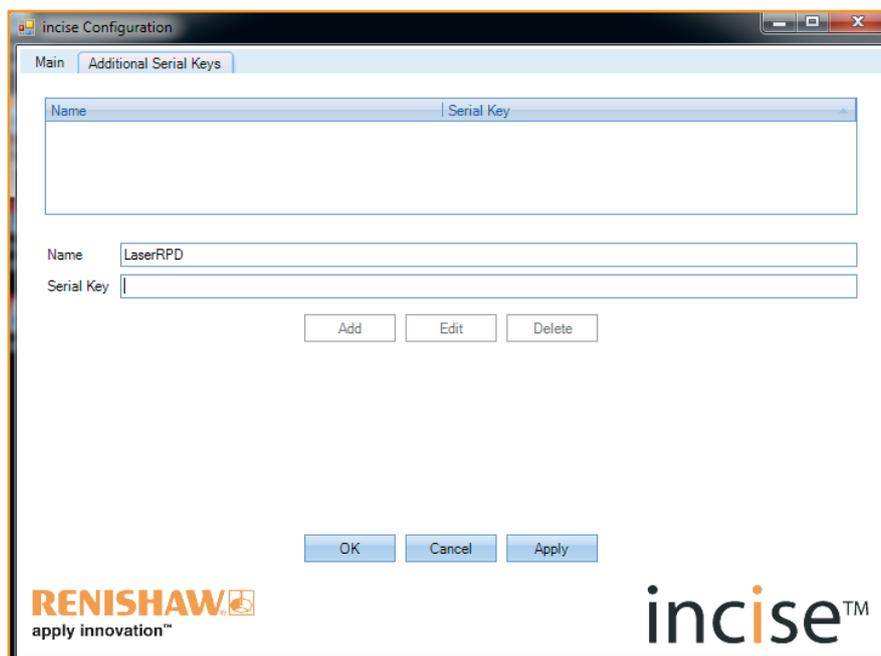

Submitting your STL



Load InciseCAD and select the  icon from the top toolbar. Input the prescription details and click “OK”.

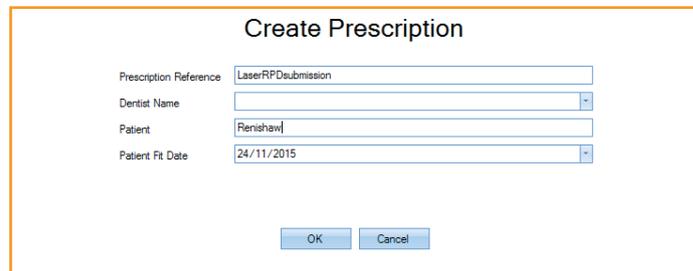

Create Prescription

Prescription Reference: LaserRPDsubmission

Dentist Name: [Dropdown]

Patient: Renishaw

Patient Fit Date: 24/11/2015

OK Cancel


Note: “Prescription Reference” will be used throughout manufacturing to identify your part. Please ensure it is traceable for your own records.

Select your prescription (now displayed on the left-hand side of the screen).



Select the  icon from the top toolbar and set the material as “LaserPFM™ (Cobalt Chrome)”.

Select any tooth from the image on the right-hand side of the screen and then select “Anatomical Units” followed by “Anatomical Coping”. Click “OK”.

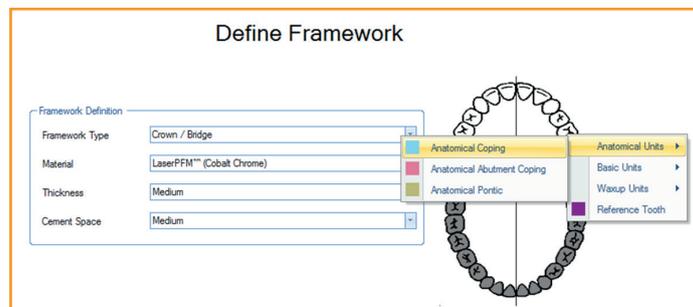

Define Framework


Framework Definition

Framework Type: Crown / Bridge

Material: LaserPFM™ (Cobalt Chrome)

Thickness: Medium

Cement Space: Medium

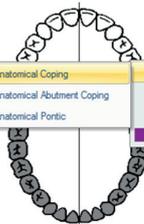

Anatomical Coping
Anatomical Abutment Coping
Anatomical Pontic

Anatomical Units
Basic Units
Waxup Units
Reference Tooth



Select the  icon from the top toolbar and upload your STL file.



Select the  icon from the top toolbar and set the “Serial Key Name” to “LaserRPD”. Click “OK”.

Select the customer serial key you wish to use for this submission.



Serial Key Name: LaserRPD

OK Cancel

Your design will now be submitted to Renishaw for manufacturing. If you have problems uploading your STL file at this stage, please contact Renishaw Dental Support on +44 (0)1453 524524.